

08/20 - USA



BROOKFIELD

INSTRUCTIONS FOR USE

Codes 6571 to 6578




CONTENTS	PAGE
1.0 INTRODUCTION	1
2.0 ILLUSTRATION & TECHNICAL DATA	2
3.0 FOR YOUR SAFETY	3
4.0 UNPACKING YOUR CHAIR	4
5.0 SETTING UP AND ADJUSTING YOUR CHAIR	5
6.0 CARE & MAINTENANCE	10
7.0 WARRANTY & SERVICE	14
8.0 CONTINUOUS IMPROVEMENT	14
9.0 SERVICE INSPECTION	15

1.0 INTRODUCTION

Thank you for choosing the Smirthwaite USA LLC Brookfield Chair.


The Brookfield is a robust chair for children from the age of 2 years old to young adult who require moderate levels of postural support when seated. Its natural wood design means that it fits wells into nursery, school and home environments.

	<p>IMPORTANT!</p> <p>These instructions should be read by all therapists and care givers using the equipment and should be retained for future reference. The product should always be used under adult supervision.</p> <p>Any incorrect use of the product and failure to follow the instructions may put the user at risk or impede the function. If you have any questions using this product or wish for further copies, please do not hesitate to contact Customer Service department on (1) 212 541 2455.</p> <p>This product has been designed and manufactured specifically and solely for use by clients with additional needs.</p>
---	---



2.0 ILLUSTRATION OF YOUR CHAIR

- A. Ramped Seat
- B. Contoured Backrest
- C. Armrest
- D. Leg Outrigger
- E. Lapstrap

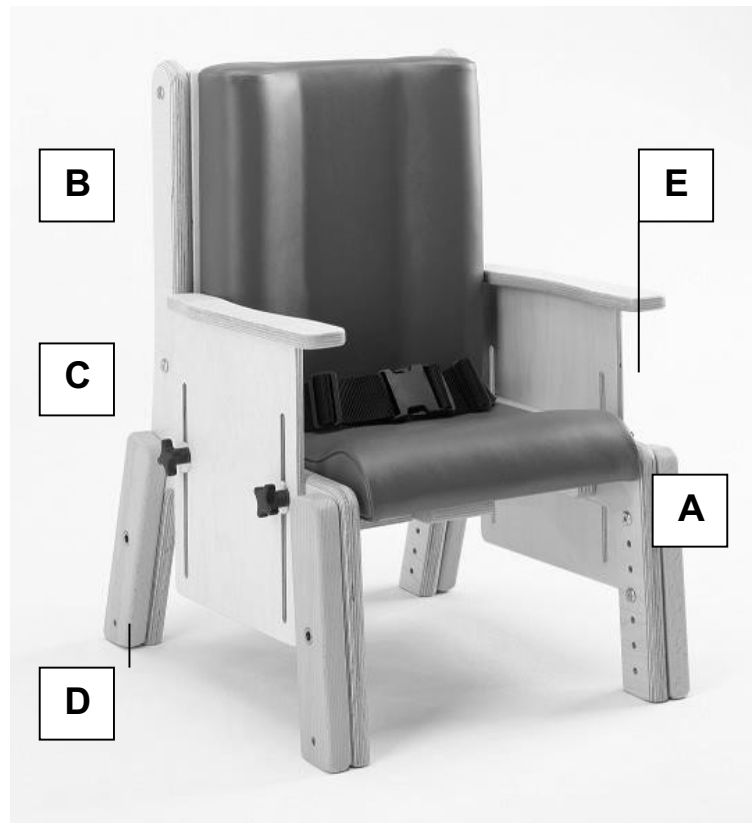

2.1 TECHNICAL DATA

SIZE	1LB	1HB	2LB	2HB	3LB	3HB	4LB	4HB
CODE	6571	6572	6573	6574	6575	6576	6577	6578
Dimension								
Seat Height (in)	9.6-12.9	9.6-12.9	11-14.9	11-14.9	13.5-17.5	13.5-17.5	16.7-22.6	16.7-22.6
Back Height (in)	14.3-19.2	18.3-23	16.1-20.8	19.4-24.2	17.7-23.2	20.8-26.3	19.8-25.3	22.4-27.9
Width (in)	12.7	12.7	14.3	14.3	17.1	17.1	17.1	17.1
Seat Depth (in)	8.6-12.5	8.6-12.5	10.8-15.1	10.8-15.1	13.9-18.3	13.9-18.3	16.7-22.4	16.7-22.4
User Weight (lbs)	121	121	143	143	165	165	187	187
Shipping Weight (lbs)	26	30	39	44	50	55	62	66



3.0 FOR YOUR SAFETY



STOP!

Please read these instructions CAREFULLY and THOROUGHLY.

- The user should **NOT** be left unattended while in the chair. Always ensure a responsible therapist or care giver is in attendance.
- If you believe the chair or any fitted accessory to be faulty at any time, **DO NOT USE** – contact Smirthwaite USA LLC by telephone on (1) 212 541 2455.
- The therapist should be familiar with the methods of adjustment and have completed all adjustments appropriately to meet the needs of the child before transferring the child onto the chair.
- The chair is **ONLY** to be used indoors on a flat level surface.
- **DO NOT EXCEED USER MAXIMUM WEIGHT LIMITS**
- Always fasten the pelvic strap and any other straps provided with the chair. When Velcro straps are used, the Velcro should overlap by at least 3in.
- Regular maintenance checks and cleaning are essential for the safe use of this equipment (see Section 6.0 Care and Maintenance).
- Always keep this product away from flames, cigarettes and sources of heat including open fireplaces, radiators, heaters.
- **DO NOT** fit parts or accessories of other manufacturers to this product unless authorized to do so in writing by Smirthwaite USA LLC. Failure to follow these instructions will not only invalidate the guarantee but could make the chair dangerous to use. Smirthwaite USA LLC will not accept liability for any injury or damage incurred through such malpractices. Any repairs required must be carried out by Smirthwaite USA LLC authorized personnel.





STOP!

LAP STRAPS & HARNESSSES SAFETY NOTICE

Lap straps and harnesses must be appropriate and safe for the user and the users clothing.

Lap straps and harnesses must be checked every time the chair is used to ensure they are fitted as prescribed by the clinician, take account of the users clothing and are tightened so that the user cannot sustain injury.

Checking the fit of lap straps and harnesses must be done with the user in the chair and should be undertaken as soon as the users sits in the chair.

4.0 UNPACKING YOUR CHAIR



STOP!

If in any doubt, ALWAYS seek ADVICE.

Always turn hand wheels, levers and screws clockwise to tighten or anti-clockwise to loosen.

- When delivered, the chair will be supplied fully assembled except for the attachment of any special accessories ordered.
- If your order has been supplied as a custom kit, please ensure you follow both the instructions documented in this IFU **and** any additional instructions supplied.
- Please take note of any instructions on the packaging/box when unpacking.
- Once the chair has been carefully unpacked, please check all parts.



STOP!

If you believe this product to be faulty – DO NOT USE – Contact Smirthwaite USA LLC on (1) 212 541 2455



5.0 SETTING UP AND ADJUSTING YOUR CHAIR



STOP!

If in any doubt, **ALWAYS** seek **ADVICE**.

Always turn hand wheels, levers and screws clockwise to tighten or anti-clockwise to loosen.



5.1 ADJUSTMENTS

Adjustments can be made to the seat; depth, seat height and back height

5.1.1 SEAT DEPTH


To move seat forwards or backwards

- Loosen handwheels (A)
- Slide seat to desired position (B)
- Allow 0.8in gap between the back of knee and the edge of the seat
- Tighten handwheels (A) when the seat depth is set to the desired position

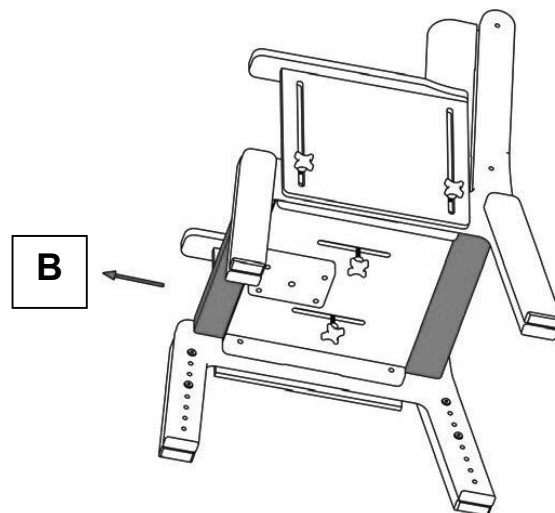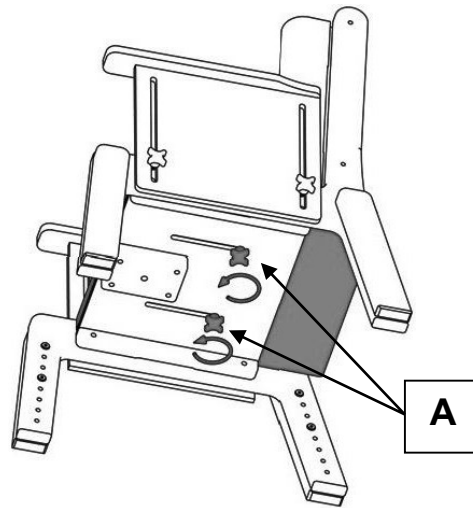

5.1.2 SEAT HEIGHT



- To adjust the seat height, reposition the leg outriggers (A) using the tool provided for this purpose.
- There are 2 screws per leg.
- Ensure the screws are re-inserted and tightened when the desired height is achieved.

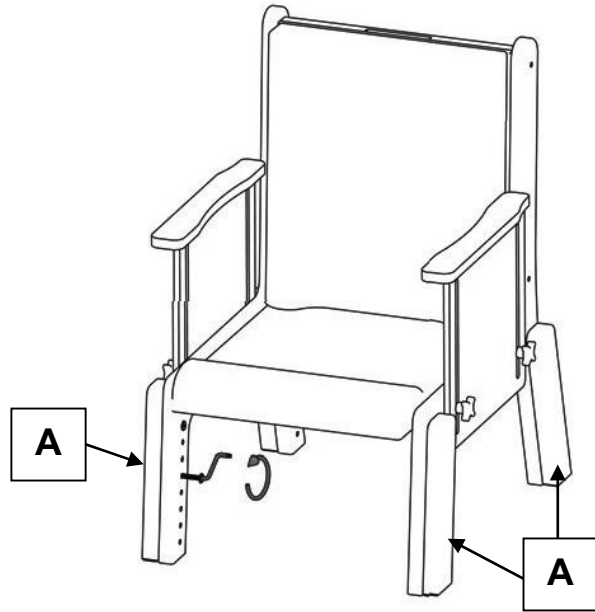

5.1.3 BACK HEIGHT

To adjust the height position of the back cushion



- Loosen handwheels (A)
- Slide back cushion to desired position (B)
- Tighten handwheels (A) when the back cushion is set to the desired position

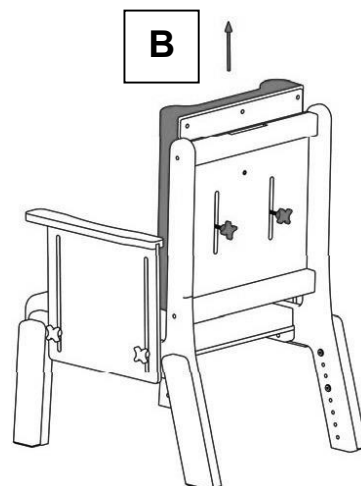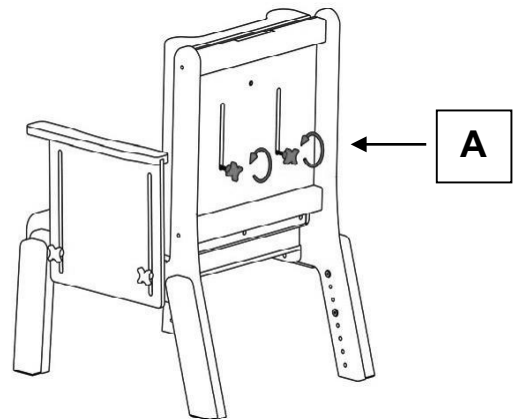

5.1.3 ARMREST HEIGHT ADJUSTMENT

To adjust the height of the armrest

- Loosen handwheels (A)
- Slide armrest to desired position (B)
- Tighten handwheels (A) when the armrest is set to the desired position
- Repeat for both armrests
- Armrests can be lowered to minimum setting to facilitate side transfer of child into the seat.

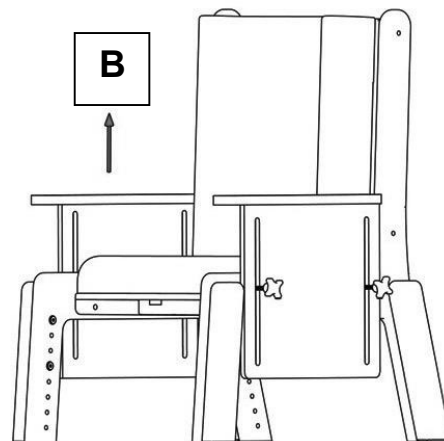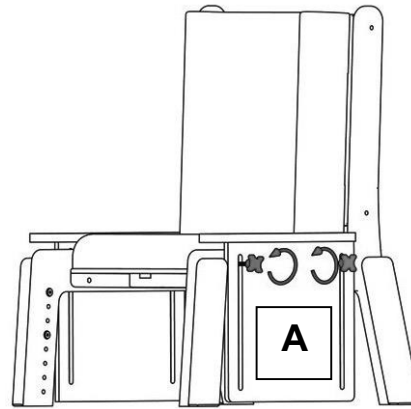


5.1.4 TRAY ACCESSORY

A wooden tray accessory is available for your Brookfield chair.

The tray slides onto the armrests, and toggles (A) are rotated to secure the tray into position.

Ensure the armrests are set to the same height before sliding the tray into place.

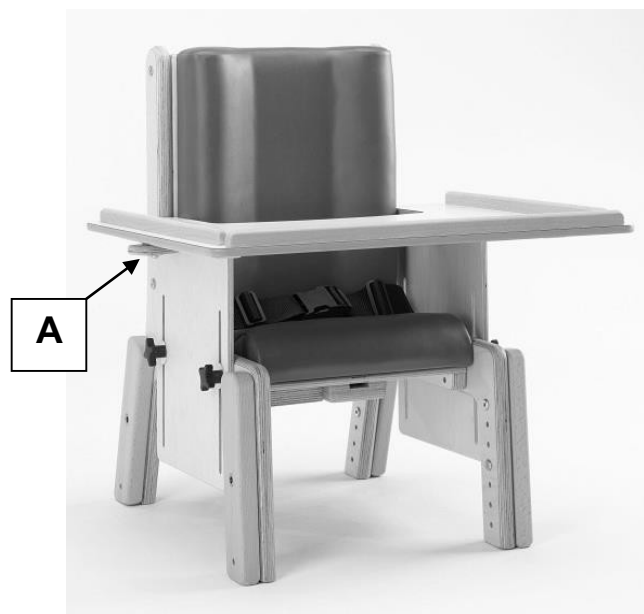


5.1.5 PELVIC STRAPS

The operation of the buckles, cam-locks and tri-glides can be found below.


5.1.5.1 PELVIC STRAPS

- Feed the two ends of the strap between the back and the base of the chair.
- Then feed the ends through the two cam-locks located underneath the base of the chair, towards the back (A).
- Secure the strap in the cam-locks ensuring you can fit two fingers between the strap and the user.

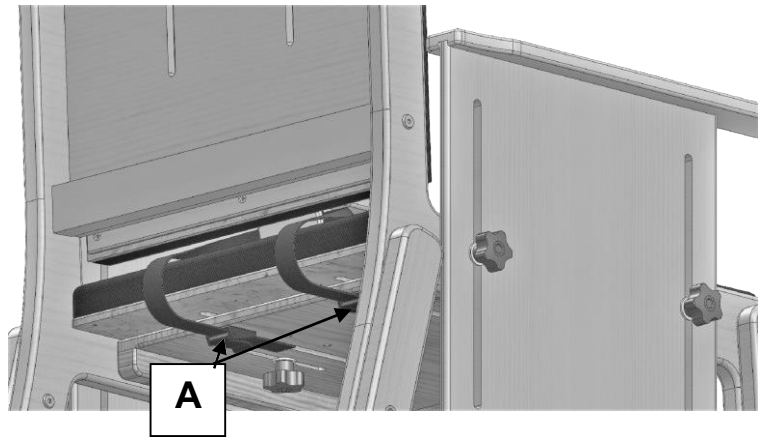

5.1.5.2 4-POINT PELVIC STRAP


- Feed the lower, shorter ends of the strap through the two slots in the base.
- Then feed these ends through the relevant cam-locks located underneath the base (A).
- Pull the upper, longer ends of the strap between the back and base of the chair.
- Feed these ends through the cam-locks located on the lower back of the chair (B).
- Secure the strap in the cam-locks ensuring you can fit two fingers between the strap and the user.

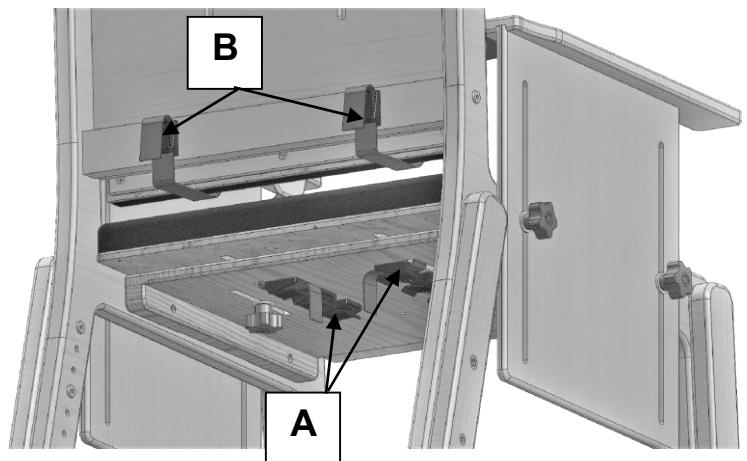

5.1.5.3 STRAP OPERATION


BUCKLE STRAP (STANDARD)

- The buckle is secured by pushing (A) into (B).
- To release the buckle, press both sides of (A)

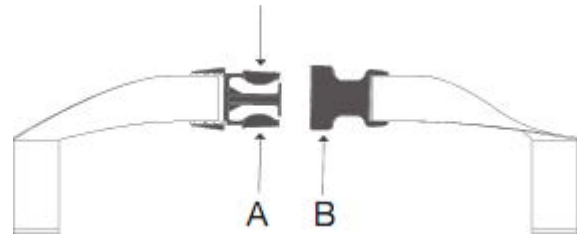


STRAP LENGTH

- The length of the straps can be altered by adjusting the amount of the strap that is fed through the cam-locks and tri-glides (if applicable). See diagrams across and below.

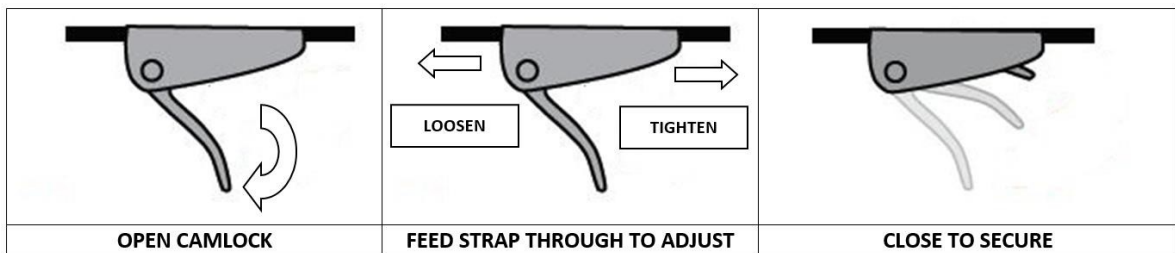

VELCRO STRAP

- The strap is secured by pressing the two Velcro sections together and is released by pulling the sections apart
- When Velcro straps are used, the Velcro should overlap by at least 3in.

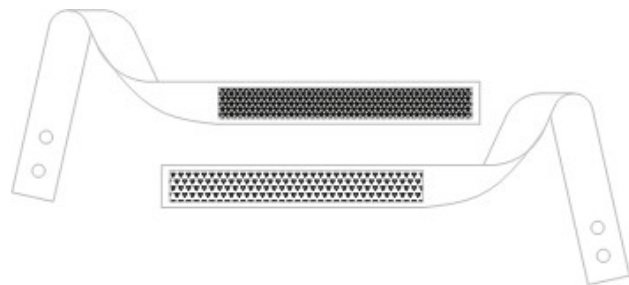


5.1.6 REPLACEMENT STRAPS

- To replace the straps, open the cam-locks and pull the straps away from the chair.



6.0 CARE AND MAINTENANCE

	IMPORTANT! CLEANING IS RECOMMENDED ON A REGULAR BASIS
---	--

- Clean upholstery and wooden parts with a damp cloth and mild detergent.
- Stubborn marks on the woodwork should be cleaned by using a soft brush.
- Do not soak or immerse the chair in water.
- The pelvic strap can be cleaned with a damp cloth. In cases of extreme soiling it can be machined washed at low temperature with mild detergent but must be drip dried.
- Store the chair in a cool dry place out of direct sunlight.
- DO NOT use bleach, solvents, abrasives, synthetic detergents, wax polishes, antibacterial sprays or wipes.

6.1 DAILY CHECKS

- Check the tray (if supplied) to ensure no parts are loose or damaged
- Check all upholstery for signs of wear and tear
- Check that the hand wheels tighten correctly, and are not cross-threaded or damaged
- Check that the leg screws are present and are tightened correctly
- Check all straps for fraying, and that buckles are not missing/damaged
- Check the strap is securely fastened to the poppers on the underside of the chair

	STOP! THE USER SHOULD NOT BE SEATED WHILE THE CHECKS ARE CARRIED OUT.
---	--

6.2 SERVICE INTERVAL

The Brookfield chair should be serviced every 3 years. Servicing must only be undertaken by Smirthwaite USA LLC authorized personnel.



6.3 NOMINAL SERVICE LIFE

Your product has a nominal service life of 12 years, during which full post-sales support will be available with regard to parts and servicing.

Product service life has been determined based upon the design complexity of product, and the anticipated exposure to normal use.

Good practice dictates all Smirthwaite USA LLC products have been designed and manufactured to high levels of safety and quality, and will meet requirements of normal use when maintained in line with our servicing recommendations.



STOP!

If the product has been out of use for an extended period of time (6 months or more) it should always be serviced prior to being reissued.

If the product has been subjected to 'heavy' or 'constant' use, the service should be reduced to half the recommended period.

Constant and/or heavy use is considered to be:

- Daily use above 7 hours duration
- Weekly use above 5 days duration
- Monthly use above 10 months per year
- Use by a client who is at 90% to 100% of the maximum weight limit of the product. The maximum weight limit must **NEVER** be exceeded
- Use by a client who is extremely active, either voluntarily or involuntarily



6.4 EXTENDING NOMINAL SERVICE LIFE

At Smirthwaite USA LLC we are proud to produce products that have a reputation for quality and durability.

We believe our products have the potential to provide benefits to our clients beyond the nominal service life documented above.

We will continue to provide full support beyond the nominal service life provided the following conditions are met:

1. A full-service schedule has been maintained.
2. A full service and inspection is undertaken at the end of the nominal service life period.
3. The product is subsequently serviced annually (or biannually if under 'heavy/constant' use conditions).
4. Smirthwaite USA LLC reserve the right to limit support where parts/components are no longer available.

6.5 DOCUMENTATION/RECORDS

- It is the responsibility of the current equipment owner to ensure the 'Instructions for Use' manual and any further manuals for accessories fitted to the equipment are handed over to the new owner at the time of exchange/sale.
- It is the responsibility of the current equipment owner to ensure the service and inspection record form is kept up to date.

6.6 PRODUCT CONFIGURATION

- Smirthwaite USA LLC will document and maintain a record of the original product configuration at the time of first sale.
- Smirthwaite USA LLC will not be held responsible for any subsequent changes to this configuration unless authorized to do so in writing by Smirthwaite USA LLC.
- It is the equipment owner's responsibility to maintain their own records of changes to the equipment configuration and to be able to provide such records to subsequent owners to maintain traceability.
- We recommend an inspection/service by Smirthwaite USA LLC authorized personnel whenever a significant change is made to product configuration to ensure the product is safe to use. **If in any doubt, ALWAYS seek ADVICE.**





IMPORTANT REMINDER!

DO NOT fit parts or accessories of other manufacturers to this product unless authorized to do so in writing by Smirthwaite USA LLC.

Any servicing or repairs required must be carried out by Smirthwaite USA LLC (or a Smirthwaite USA LLC trained engineer).

If you believe this product to be faulty – DO NOT USE – Contact Smirthwaite USA LLC on (1) 212 541 2455.

If in any doubt, ALWAYS seek ADVICE.



7.0 WARRANTY

Smirthwaite warrants the products detailed on your order to be free from defects in materials and workmanship for a period of 2 years from date of delivery. If a fault develops during this period, please contact Customer Services team by email info@smirthwaite.com or phone (1) 212 541 2455 who will advise you on the best course of action.

The warranty excludes faults due to accident, neglect, misuse, not following the Instructions and normal wear and tear. This guarantee is in addition to your legal rights. A charge may be made where the goods cannot be repaired under the warranty. You will be advised before this is made.

8.0 CONTINUOUS IMPROVEMENT

Smirthwaite USA LLC is committed to continuous improvement to their product range. Should you have any suggestions or comments please send them to our product design department at: info@smirthwaite.com

Smirthwaite USA LLC reserve the right to change the specification or material without prior notice.

For catalogs, help and further information on our products please contact us at:
Smirthwaite USA LLC, 405 Lexington Avenue, 26th Floor, New York, NY10174
T. (1) 212 541 2455 E: info@smirthwaite.com



9.0 SERVICE INSPECTION

9.1 Product Information

Model	
Size	
Date of Manufacture	
Serial Number	
Final inspection	

9.2 Service & inspection record form:

Date	Procedure	Service Personnel





Smirthwaite USA LLC, 405 Lexington Avenue, 26th Floor, New York, NY10174
T. (1) 212 541 2455 E: info@smirthwaite.com www.smirthwaite.com

