

LAY ON SIDE, PULL OUT




BOTTOM

****DO NOT TILT THE BOX UPSIDE DOWN AND TRY TO USE GRAVITY TO REMOVE THE CHAIR, IT WILL RESULT IN DAMAGE TO THE CHAIR****

- Open the top of the box as received and remove the foam lid inside the package
- To remove the power chair, orientate the box such that when it is laid down on the ground the padded bar is at the top and the footplate is at the bottom
- Grab the top bar and begin to pull your Travel Buggy out horizontally parallel to the ground

UNFOLD & SECURE BACK LOCK

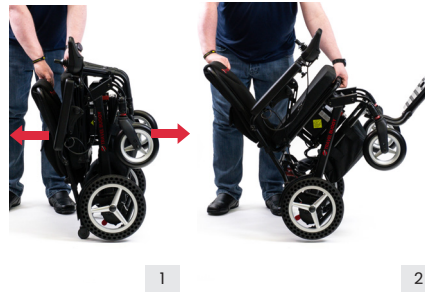

- Open the DASH like a lawn chair by pulling the top bar and seat away from each other
- Continue pressing down on the seat until the chair is on its 4 wheels
- You'll here a click when the DASH LITE locks into the unfolded position

WARNING: Never transport the DASH LITE in a tie down system, as it is not compatible with them. Never sit in the power chair and be transported in a moving vehicle. Do not store or transport the folded DASH LITE in the front passenger seat with the driver where it could move or slide.

TURN BATTERY ON



- Remove the battery from under the power chair by pulling the red release lever away from the battery and simultaneously pulling the battery towards you.
- **IMPORTANT!** Turn the battery to the 'ON' position and insert it back into the chair by sliding it in to the slot under the chair

INSERT THE JOYSTICK



- Locate the joystick controller in the storage pouch below the power chair and slide it into the armrest tube until a comfortable arm length is achieved for the user (image 1).



- Tighten the corkscrew to secure the joystick (image 2)
- Align the arrows of the joystick connection cables and press them together to plug the joystick controller in (image 3)



- Rotate the coupling over the connection point to secure the connection point (image 4)

GETTING IN AND OUT



- Before sitting in the Power Chair, be sure the power button is OFF and the rear brake levers are in the DOWN (locked) position
- If you prefer a front entry, we recommend flipping up the foot plate
- DO NOT step on the footplate when entering or exiting DASH LITE as it may cause the power chair to move in an unsafe manner
- To enter on the side push the arm release button as shown in image 3. Raise arm rest back and away. When seated, lower arm rest back into position and ensure it is locked

CONTROLLER

+ INCREASE SPEED

Press the PLUS + button to increase the speed

- DECREASE SPEED

Press the MINUS - button to decrease the speed

Horn

Press the  button to sound the horn

5-Speed Control Lights

SPEED CONTROL LIMITS

- 1-Light On 1.5 MPH
- 2-Lights On 2.25 MPH
- 3-Lights On 3 MPH
- 4-Lights On 3.75 MPH
- All 5-Lights On 4.5 MPH



⏻ POWER ON

Press the GREEN button to turn ON the power chair

⏻ POWER OFF

Press the RED button to turn OFF the power chair

Battery Indicator Lights

There are a total of 7 battery power indicator lights: 3 Green, 3 Yellow and 1 Red. From top down the lights will disappear as the battery drains indicating the amount of charge remaining. Recharge at any point before the last Red light disappears, indicating a fully depleted battery.

- DASH Lite Power Chair has an easy to use 360 degree joystick controller featuring multiple speeds and a battery life indicator
- Take some time in a safe, controlled environment to learn the operation of the controller, the speed and the maneuverability of the DASH Lite
- Be sure to adjust the speed depending on your environment and while turning
- To slow down, slowly bring the joystick back toward the initial position.
- To stop, just bring the joystick back to the initial position
- Use extra care when you drive the DASH Lite in reverse

FREE WHEEL MODE



- To put your DASH LITE in Free Wheel Mode, come to a full stop and turn OFF the power on the controller
- Push UP on the two red levers located on each rear wheel as shown