# QUICK REFERENCE GUIDE HAND CONTROL

**NOTE:** This quick reference guide offers basic information on only hand control components and functions. Read the owner's manual and all other supplemental information thoroughly before operating the chair for information on safety, basic operation, and care and maintenance.

The illustrations used reflect those hand controls with the highest level of functionality. The hand control you receive may have fewer options, but basic functionality is the same.

The hand control on your chair may be equipped with an LED that indicates power to the hand control. If this LED is present and not lit when the hand control is in use, contact your authorized Provider.

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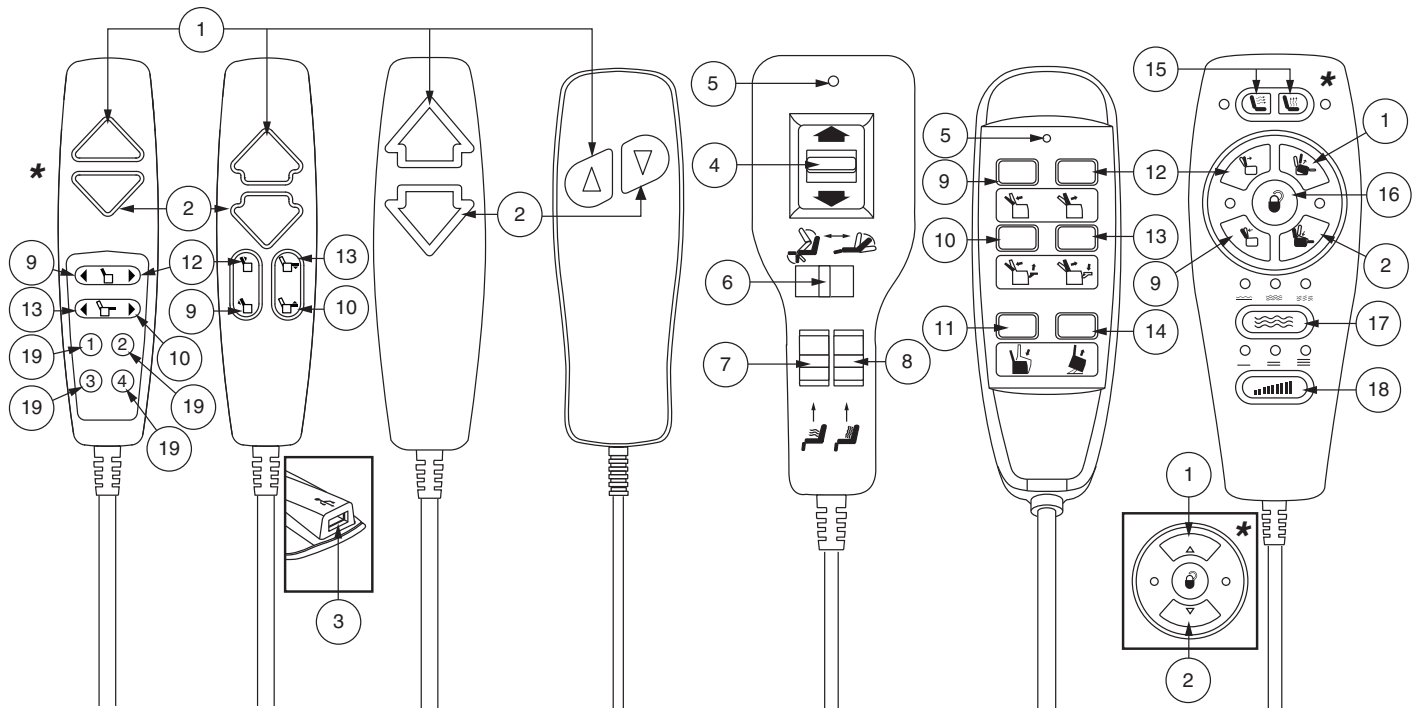
Rev H/January 2019

**1 UP BUTTON**  
Press and hold to raise chair to seated or standing position.

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**2 DOWN BUTTON**  
Press and hold to lower chair to seated or reclined position.  
  
*NOTE: On infinite position models, only the footrest is extended into a reclined position when this button is pressed. The backrest remains upright.*

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**3 USB PORT**  
Use for USB compatible devices.

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**4 UP/DOWN SWITCH**  
Controls seat lift and recline positions. If equipped with INHIBIT SWITCH, also controls footrest.

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**5 POWER LED**  
Indicates power to the hand control. Will only light when the hand control is in use (any button pressed to adjust chair).

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**6 INHIBIT SWITCH**  
If switch is set to the left, the UP/DOWN SWITCH controls the lift position of the chair and footrest. If switch is set to the right, the UP/DOWN SWITCH controls the recline position of the seatback.

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**7 HEAT SWITCH**  
Slide to the up position to activate the heat option. Slide to the down position to deactivate the heat option. Will shut off automatically after 20 minutes.

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
**8 MASSAGE SWITCH**  
Slide to the up position to activate the massage option. Slide to the down position to deactivate the massage option.

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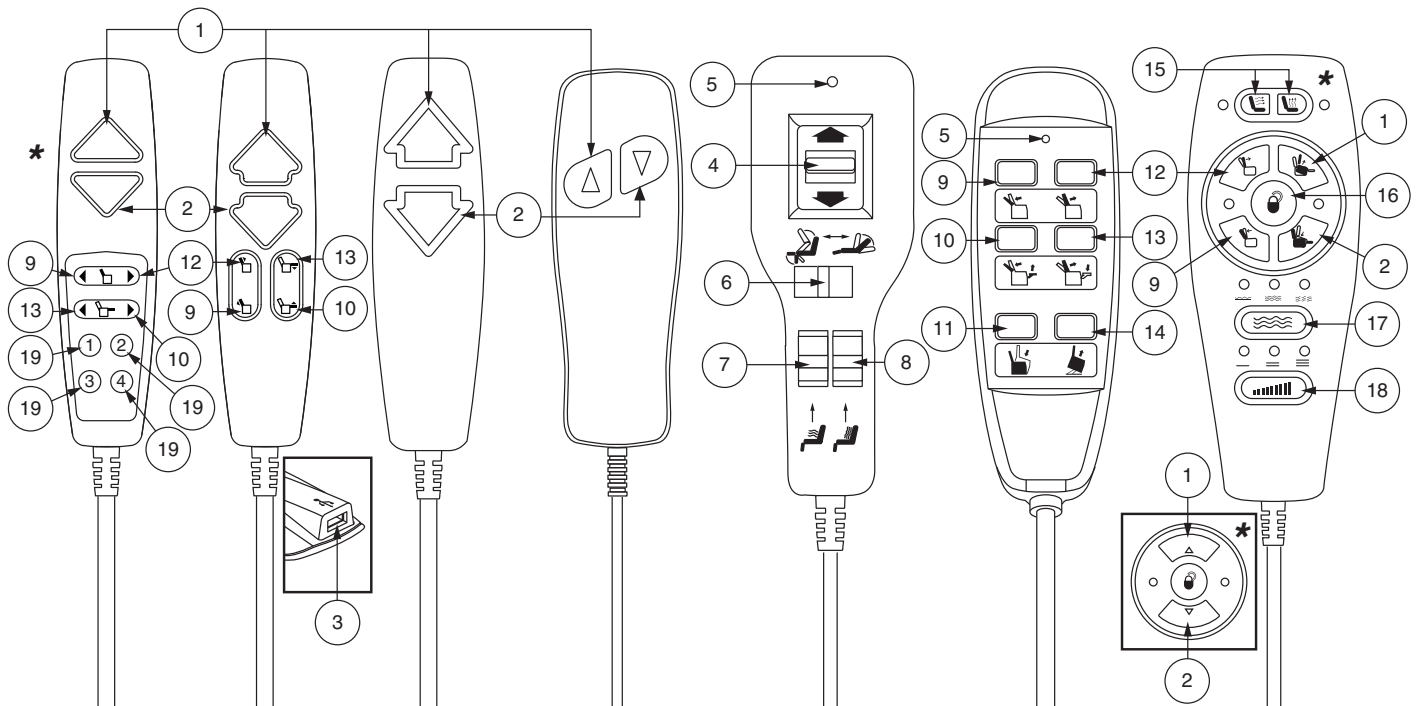**15 HEAT BUTTON (SEATBACK) AND (SEATBASE)**  
 Push once (1x) for low heat—green LED.  
 Push again (2x) for high heat—red LED.  
 Push again (3x) to shut off—no LED.  
 Will shut off automatically after 20 minutes.

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**16 LOCK/UNLOCK BUTTON**  
 Press and hold 3 seconds to lock. Right LED will illuminate red. Press and hold 6 seconds to unlock. Left LED will illuminate green.

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**17 MASSAGE BUTTON WITH MODE**  
 Push once to activate massage option. Push again to cycle through modes indicated by LEDs above button (wave, steady, or pulse). Will shut off automatically after 20 minutes.

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**18 INTENSITY BUTTON**  
 Push to cycle through the 3 intensities for the massage option indicated by the LEDs above button (low, medium, and high).

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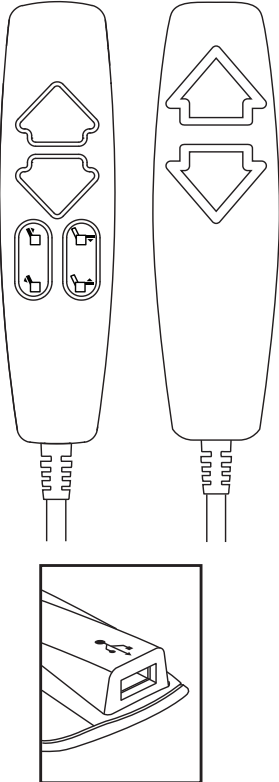
**19 MEMORY BUTTONS**  
 Press and hold Memory Button 1 and Memory Button 2 for three (3) seconds. The LEDs on the memory buttons will flash to indicate that the hand control is in "set" mode. Press and hold the desired memory button for three (3) seconds to set the memory position. The LEDs will stop flashing.  
 To go to a preset memory position, press and hold the desired memory button.

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**\*NOTE: For hand controls equipped with memory buttons, flash codes are not applicable. However, the USB port will still become disabled once the chair enters low power mode. If your hand control is equipped with memory buttons and the USB port stops working, contact your authorized Pride Provider for assistance.**

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