

CHALLENGER BEACH CART ASSEMBLY INSTRUCTIONS



Insert the wheel axle through the body of the frame. You will notice the axle has three holes. With your frame positioned as pictured above, slide the side of the axle that has the two holes into the frame. The side with the two holes must be on the right side of your cart.

You will now insert one of your cotter pins into the inner most axle hole making sure it goes through the body of the frame, as pictured on the next page. Be sure to bend the cotter pins so they do not fall out.



**Now simply install both wheels, with the valve stems facing outward.
Insert the remaining cotter pins to each side.**



Your installation is complete.