

# Scooter Walker Holder

## JWH

Carry any Folding Walker and most Folding Rollators with a cross-bar. Use the 3 Velcro straps to secure walker to carrier. Virtually any walker/Rollator weighing less than 25 pounds.

### PARTS



Clevis Pin (2)



Tension Knob (2)



Walker Holder Upper



Walker Holder Mount



Walker Holder Vertical

### ASSEMBLY INSTRUCTIONS

Take the Mount Bar in hand with the holes horizontal and the L shape pointing up.




Slide the Vertical Bar onto the Mount Bar, until the lower holes line up (either way).



Insert the Upper Bar into the Vertical bar. Secure this in place with a Tension Knob (either way).



Adjust the Vertical Bar to the center of the Mount Bar. Pin this in place with a Clevis Pin. Secure this in place with a Tension Knob.

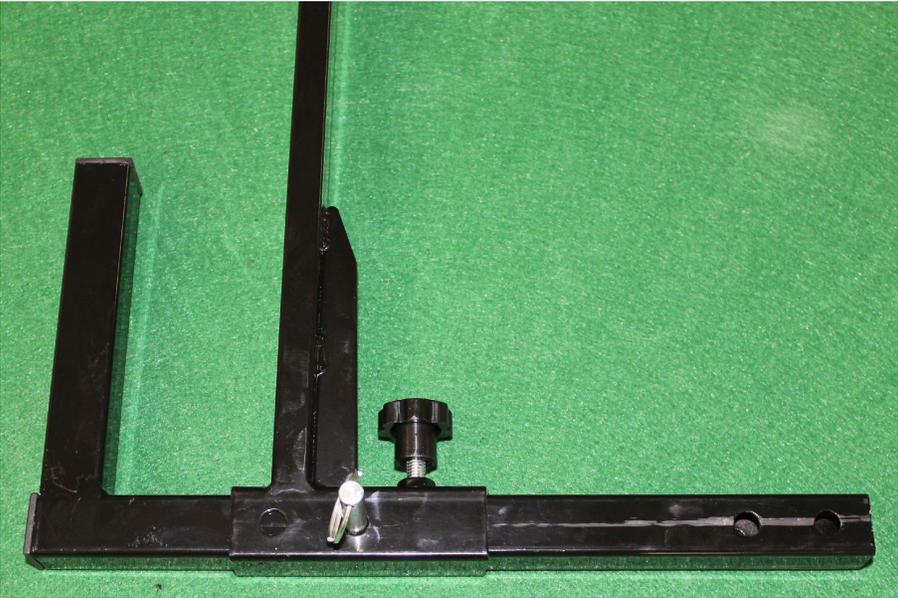

Slide the Mount Bar into the receiver under the seat. Pin this in place with a clevis Pin using the first or second hole.



Use the Velcro in the center of the Vertical bar to keep the Walker from bouncing.

Unwrap the Velcro Straps at the top of the Walker Holder.

Place your walker onto the upper Bar. Replace the Velcro Straps.