



## **Strive Adaptive Stroller Accessories**

Canopy (ST8100S, ST8100L)

Headrest Extension (ST8112, ST8114, ST8116, ST8118)

Padded Winged Headrest (ST8801)

Bus Transit Tie-Downs (ST8802)

Basket (ST8803)

Anti-Tippers (ST8804)

Upper Extremity Support (ST8805)

Seat Depth Extension Kit (ST8806)

## Canopy (ST8100S, ST8100L)

Extendable canopy with a clear flap-covered window. Provides protection from the rain and sunlight.

1. Snap onto the vertical pipes at the desired height (*Fig 1*).



Figure 1

## Headrest Extension (ST8112, ST8114, ST8116, ST8118)

Extends back height and support for taller individuals.

1. Slide the Headrest Extension posts into the pipes attached to the back of the stroller (*Fig 2A*).
2. To adjust height, adjust the position of the two round rubber washers on the headrest (*Fig 2B*).
3. To adjust tension, use the four hook and loop fastener straps on the back of the headrest (*Fig 2C*).



Figure 2

### **Padded Winged Headrest (ST8801)**

Adjustable padded headrest provides comfortable midline positioning, promoting optimal posture and alignment.

1. Thread the cords through four grommets in the back and/or headrest extension at the desired height (*Fig 3*).
2. Tie the cords together in the back.

To adjust the Padded Winged Headrest's position, thread the cords through different grommets.

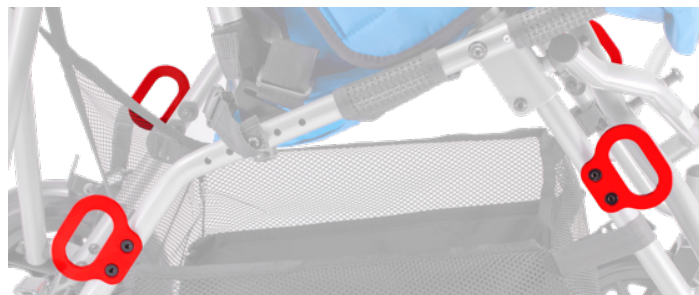


*Figure 3*

### **Bus Transit Tie-Downs (ST8802)**

Secures stroller for safe transportation in a bus/van.

1. Take each tie-down assembly out of the box.
2. Remove the lock nuts and bent metal washers (keep the plastic pieces on the bolt).
3. Attach the tie-downs to the stroller legs (*Fig 4*).
4. Fasten with the washers and lock nuts.
5. Use the included wrench and small hex key to tighten every bolt.



*Figure 4*

## Basket (ST8803)

Under-seat storage basket.

1. Attach the basket to the stroller using all five straps as shown below (*Fig 5*).
2. Tighten straps.
3. Test to make sure straps will hold.



*Figure 5*

## Anti-Tippers (ST8804)

Prevents backward tipping and enhances stability and security.

1. Remove the rubber stop on the curb-assist lever (might need to hammer or pry off)
2. Insert the anti-tippers (make sure the locking pin locks) (*Fig 6*).



*Figure 6*



## Upper Extremity Support (ST8805)

Depth adjustable, multi-functional clear tray is easy to clean and fits all sizes.

1. To assemble the tray, slide the two metal plates into the slots on the bottom (*Fig 7A*).
2. Undo the hook and loop fasteners on both side panels (for access) (*Fig 7B*).
3. Using the included wrench and medium sized hex key, partially loosen the bolt (*Fig 7C*).
4. Slide the slot of the mounting bracket around the bolt between the washer and pipe (*Fig 7D*).
5. Snap the other end of the tray mounting bracket onto the stroller frame (*Fig 7E*).
6. Repeat steps 3-5 on the other side.
7. Retighten the bolts.
8. Slide the metal rods into the mounting brackets.
9. Tighten the tray screw (*Fig 7F*).
10. To lock the tray in place, tighten the hand knobs on the mounting brackets.
11. Refasten the hook and loop fasteners on both side panels (*Fig 7*).



Figure 7

## Seat Depth Extension Kit (ST8806)

1. Undo the hook and loop fastener that attaches the back panel to the seat panel (*Fig 8*).
2. Relieve fabric tension by partially folding the chair.
3. Pull the seat panel off the seat rail in the front (both sides).
4. Unscrew the bolt on the side of the seat rail (*Fig 8A*).
5. Pull out the original seat tube.
6. Insert the Seat Depth Extension Kit seat tube.
7. Move new seat tube to the desired hole.
8. Refasten the bolt.
9. Repeat steps 4-8 on the other side.
10. Pull the seat panel back over the seat rail in the front.
11. Reattach the hook and loop fastener.
12. Fully unfold the chair.



Figure 8